**ZeroPlus®**  
Strong

# KOMPRESÖRLÜ NEBÜLİZATÖR

Model  
KS-NEB-001

**Türkçe**

# Kompresör Nebülizatör

## Model KS-NEB-001

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**Model  
KS-NEB-001**

**English**

The logo for ZeroPlus Strong features a stylized graphic of four dark, rounded shapes arranged in a cluster to the left of the text. The text 'ZeroPlus' is in a large, bold, sans-serif font, with a registered trademark symbol (®) to its upper right. Below 'ZeroPlus' is the word 'Strong' in a smaller, bold, sans-serif font.

# **ZeroPlus<sup>®</sup> Strong** COMPRESSOR NEBULIZER

## **Model KS-NEB-001**

### **ZEROPLUS STRONG COMPRESSOR NEBULIZER**

#### **User's Manual**


Please, read the instructions carefully before operating this product

#### **1. What is Nebulizer?**

Medical Air Compressor Nebulizer System is a medical device designed to administer the treatment to the lung pathways and prescribed by a physician. Combined Nebulizer and air compressor provides aerosol mist for inhalation of the patient and only for medical treatment.

#### **2. How does nebulizer operate?**

This unit is compact, easy to start and appropriate to move. Single cylinder piston pump, oil-free lubricant is used to create Air Pressure. Air passes to the air compressor unit of Nebulizer through air filter, to the jet outlet and then through Nebulizer. Air compressor unit and air flow from here are shown in Figure 1.

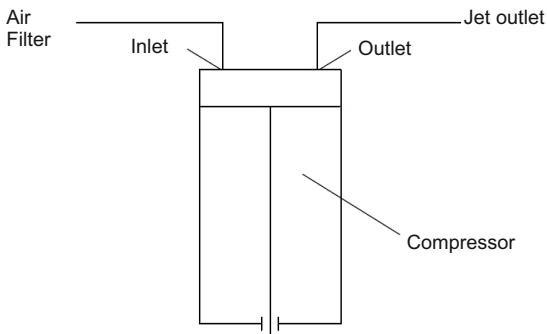**Figure 1: Operating Principle**


### **3. Control list for opening the package**

Make sure that following materials are available before using the product:

- 1x Main device (Compressor)
- 1x Aerosol Nebulizer
- 1xAeresol Tube system
- 1x Mask (Child & adult )
- 1x Mouthpiece

### **4. Box Content**

Parts of Standard Nebulization have been provided in a diagram in Figure 2.

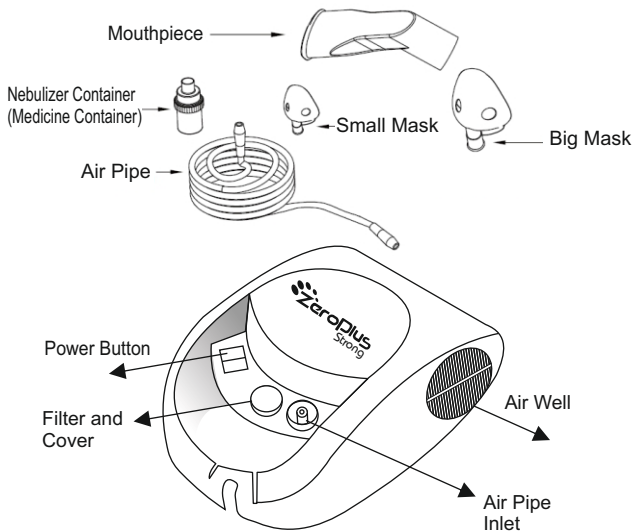
**Figure 2**

### **5. How will I put the medicine into the Nebulizer?**

To put the medicine into the Nebulizer, follow the steps below:

1. Open the cover (See Figure 3)
2. Put the medicine as prescribed by your physician into the container. Make sure that medicine does not exceed the liquid area 6. ml(6cc).
3. Close the cover.
4. Fasten the mouthpiece.

Note: Mouthpiece will be used only for oral inhalation.

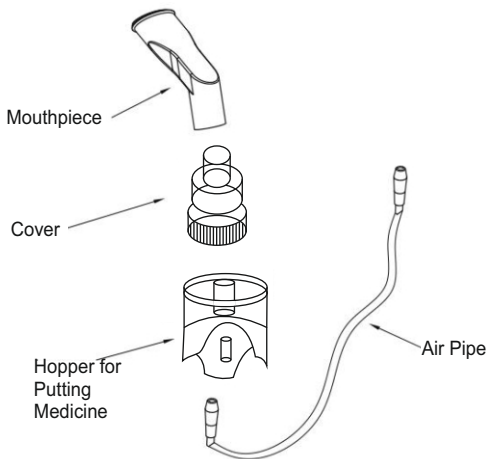**Figure 3**

## **6. How will I start the Nebulizer?**

To start the Nebulizer, follow the steps below:

1. Plug in the air filter but do not connect the main Nebulizer.
2. Make sure that the Air Filter is connected to the air compressor.
3. Turn on the power button.
4. After the air compressor is started properly, connect the jet outlet of the air compressor to the inlet of the nebulizer by the air pipe.
5. Start the treatment according to the instructions of your physician.
6. Respire naturally. Following completion of the treatment, make sure that the device is stopped.
7. Following use of the device, close the air compressor unit before separating the air pipe.

**WARNING: Do not use the product near combustible materials or explosive materials.**

## **7. What will I do after using the Nebulizer?**

To ensure that the Nebulizer is started properly, follow the steps below after each use:

1. Remove the mouthpiece after removing the air pipe.
2. Open the cover and empty the container.  
Wash it in running water for 15 minutes or keep it in warm water. Put some vinegar into the water for more detailed hygienic cleaning.

Note: Don not boil the Nebulizer. This may damage the device.

3. Dry the Nebulizer completely before storing it.

## **8. How will I change the Air Filter?**

Generally, air filter must be replaced in every three months or after use for 300 times. To replace the air filter:

1. Remove the filter cover.
2. Replace the filter.
3. Insert the filter cover.

Note: In each case where the filter gets wet, dry it well before using.

## **9. What kind of maintenance is necessary to store the Nebulizer?**

Always use the Nebulizer carefully.

When the Nebulizer is not used, unplug the device.

Keep the Nebulizer out of reach of children.

In case you store the Nebulizer, do not bend the air pipe.

In case you store the Nebulizer, keep it in the following ambient temperatures:

- 1) **Temperature: -40°C-55°C**
- 2) **Relative Humidity: 595%**
- 3) **Atmospheric Pressure: 0.05 MPa- 0.106 MPa.**

**If possible, keep the Nebulizer in well-ventilated place.**

## 10. Properties

<b>ACCESSORIES SET</b>	6 ml Medicine Cup
	2 Meter Air Tube
	2 Pieces Masks
	Filter (5 Pcs.)
	Mouth Piece

<b>SPECIFICATION</b>	Power / Frequency	AC ^ 230 V / 50-60 Hz
	Current	170 VA / 0,7 A
	Medication Capacity	6 ml
	Particle Size	0,4 to 9 $\mu$ m
	MMAD	2,79 $\mu$ m
	GSD	2,57
	Average Nebulization Rate	0,07 ml / min
	Noise Level	Below 70 dBA
	Compressor Pressure Range	2,8 -3,5 bar
	Operation Temp.	32,2 C
	Compressor	8,12 l / dak (min)
	Compressor Out of Pressure	8 psi
	Aerosol Supply	0,08 ml /dak (min)

## 11. Troubleshooting Manual

Sign	Probable Cause	Solution
Excessive Sound	No filter in the unit	Put a filter into the unit
	Main Compressor button is turned off	Turn on the unit
	Nebulizer has not been cleaned after previous use	Clean the unit
	Air pipe bent.	Remove bending and entanglement in air pipe
	Filter blocked	Remove the filter.
	No medicine available	Add sufficient amount of medicine specified by the physician to the container.
	Cover not properly closed.	Adjust the cover again.
Water drops in the air pipe	Excessively filled	Add appropriate amount of medicine to the container, connect the air pipe to the air compressor and open the unit.
	Nebulizer has been washed but not dried.	Turn on and off the Nebulizer with your finger over and over again until the drops disappear.

**Note: Please, contact with your supplier for all problems.**

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[www.medisec.com.tr](http://www.medisec.com.tr)

**Zeroplus Strong**  
**Compressor Nebulizer**  
**MODEL NO: KS-NEB-001**

**Lower Respiratory Tract**  
**Particle Size**  
**Adult and Child Mask**  
**6cc Medicine Hopper**

**MINISTRY OF INDUSTRY AND TRADE**  
**DIRECTORATE GENERAL OF CONSUMER PROTECTION AND MARKET**  
**SURVEILLANCE**

### **CERTIFICATE OF WARRANTY**

Issue Date and Number of This Certificate:

Validity Date of the Certificate:

Use of This Certificate of Warranty has been allowed by the Directorate General Of Consumer Protection And Market Surveillance of T.R. Ministry of Industry and Trade in accordance with the Law No.4077 Regarding Consumer Protection and the Regulation Regarding Implementation Procedures of Warranty Certificate come into force based such Law.

Manufacturer's / Importing Company's:

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Telefax :+90 342 220 97 35

[www.kaslarsaglik.com.tr](http://www.kaslarsaglik.com.tr)

Signature-Seal of the Authorized Person of the Company:

Product Description	: Nebulizer
Brand	: Zeroplus Strong
Model	: KS-NEB-001
Banderole and Serial No.	:
Delivery Date and Place	:
Warranty Period	: 2 Years
Maximum Repair period	: 30 Days

## **WARRANTY CONDITIONS**

1. Warranty period will start from delivery date of the goods and is for 2 (two) years.
2. All goods including relevant parts of the goods are within the scope of our company's warranty.
3. In the event that the goods fails within the warranty period of the goods, repair period will be added to the warranty period. Repair period of the goods will be maximum 30 (thirty) working days. This period will start from the communication date for the failure of the goods to the service station, if such a service station is not available, to the seller, vendor, agent, representative office, importer or manufacturer, producer of the good. In the event that said failure cannot be removed within 15 working days, manufacturer-Producer or importer must make another goods with the same properties available for use of the consumer until repair work of the goods is completed.
4. In the event that the goods fails due to material and workmanship or assembly faults within the warranty period, such goods will be repaired without charging any fee under the name of workmanship fee, fee for replaced part or any other cost.
5. Although the consumer uses the right to have the goods repaired; if :
  - the same failure occurs more than twice or different failures occur more than four times within one year or total number of different failures is more than six within the warranty period specified and such failures make the use of the goods impossible continuously as from the date on which the goods is delivered to the consumer provided that such failures occur within the warranty period specified;
  - maximum period for repair is exceeded;
  - the service station of the concerned company is not available and in the event that such failure cannot be removed with repair work according to a report to be issued by respectively, seller or vendor, or agent, representative office, importer or manufacturer-producer, consumer may demand free replacement of the goods, refund or discount in proportion to the defect of the goods.
6. Failures arisen from improper use and against the conditions set forth in the user's manual will be outside the scope of the warranty.
7. Application may be made to the Directorate General of Consumer Protection and Market Surveillance of T.R. Ministry of Industry and Trade in respect of the problems related to the Certificate of Warranty.

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Title:Medisec Tıbbi Cihaz ve Ürünleri ithalat ihracat San.ve Tic.Ltd.Şti
Kaşlar Sağlık Ürünleri Paz. Tic. Ltd. Şti.